Shower & Bath

Rubis

Hygiene, comfort and safety for bed bound individuals

Mobile Shower System

REF. 3017.00





Shower & Bath

Rubis

Step 1



Firstly, fill **Rubis's** water container

Step 2



Position **Rubis** at the end of the bed and select temperature. Press the heater button and wait for **Rubis** to reach desired temperature (this takes approximately ten minutes from cold)

Step 3



Lay one half of the purpose designed, double-width towel over the mattress, and allow the other half to rest over the side rails.

Step 4



Next, fit the waterproof sheet under the patient. Position the sheet as evenly as possible using the fasteners provided, and fix to side rails and bed ends to ensure water containment.

Step 5



Unclip the shower hose in readiness to use. Then unclip the water vacuum hose and position it at the lowest point of the mattress where water pools fastest. Remove the patient's clothing and check the waters output temperature prior to application.

Step 6



Vhen tha water temperature has reached he desired pre-set temperature it can then e applied to the patient.



With the shower procedure complete the patient can be dried and the sheet removed and stored. The patient can then be dressed.

Step 8



With the hygiene procedure complete **Rubis** can then be emptied of waste water, and cleaned in readiness for the next use.



Shower & Bath













UNIQUE FEATURES

SUSTAINABLE WELL-BEING

- · Large capacity stainless steel tanks.
- Generous water flow and powerful drainage pump.
- IPX4 adjustable water controller.
- Anti-scald maximum temperature pre-set

- Thermic disinfection procedure.

Mobile Shower System

REF. 3017.00





- 1 Control panel
- 2 Waste water tank
- 3 Fresh water tank
- **4** Single handle bar faucet (option 3017.60)
- 5 Washbasin (option 3017.60)
- 6 Stainless steel bowl (option 3017.60)
- 7 Stainless steel handrail
- 8 Maintenance access panel
- 9 Disinfectant tank access panel
- 10 Wheel with brake
- **11** Flow intensity adjustment valve
- 12 Cleaning shower
- 13 Fresh water tank drainage
- 14 Waste water tank drainage
- 15 Mattress water drainage hose
- 16 230V electric feeding cable

The Rubis Shower in Bed System comprises: a mobile shower trolley with six directional wheels, flexible showering hoses, fresh water tank, integral waste water tank and disinfectant tank, waterproof bed sheet (width 90cm), four fasteners, as well as a bath towel (158x220cm).

Reval® Continuing Care Limited Unit 17 Brynmenyn Busines Centre St Theodore's Way Brynmenyn CF32 9TZ BRIDGEND - WALES Tel: 01656 721 001 Fax: 01656 720 006 www.revalcc.co.uk

DIMENSIONS:

Overall: Height 106 cm / Width 94 cm / Depth 58 cm

DESCRIPTION:

- Steel frame, white colour, anti-corrosion coated.
- Covers in ABS PMMA
- x6 Ø100mm directional wheels, 2 with brakes.
- 2 stainless steel handrails.
- Fresh water tank in stainless steel, capacity 23L.
- Waste water tank in stainless steel, capacity 23L.
- 24V shower pump, 8 litres/min
- 24V drainage pump, 14 litres/min
- Simultaneous pump function
- Temperature controller with double display
- (real temperature & set temperature)

- Integrated differential circuit breaker 16A 30mA.
- Electric components inside IPX5 waterproof box.
- IP68 waterproof light buttons.
- Handheld shower with flow adjustable trigger.
- Concealed disinfectant tank.
- Silicone autoclave piping.

DESCRIPTION OF SHOWER SHEET:

- 13% PES 87% PVC
- 400gr/m²



CONFORMITY:

(E

• 93/42/CEE Medical Devbice - Classe 1 • IPX4 - Class 1 - CEI 60601 • CE marking

SUPPLIED ACCESSORIES:

- 3017.10 Shower sheet for bed 90cm
- 3017.20 White Bath towel 158x220cm 100% coton 400gr/m² Four fasteners

- - Red warning light (water too hot).
 - 230V 2700W Heating element
 - Water sensor level for heating element.

- Fire standard : ISO14116: 1



Shower & Bath

Rubis

3017.10 / 3017.11



90 x190cm shower sheet for bed 120 x 200cm shower sheet for bed Material: polyurethane & woven fabric.

17.12



ingle use shower sheet for bed.

7.20

8 x 200cm bath towel 0% cotton 400gr/m²

17 50

ADDITIONAL ACCESSORIES



ubis tap filling adapter set. | adapter, 1 push connector & | x 1m50 flexible)

3017.5



Complete shower flexible wi



301



Fastener.

G

iet of 2 dispensers: 1 for the hampoo & 1 for the washing broduct.

3017.55



304L stainless steel basket.

3017.60



"Patient's toilet" Option including:

- Washbasin with integrated drainage & siphon.
- Single handle bar faucet.
- 4.5L stainless steel bowl
- Possible adaptation of the single use bowl (ref.3017.56)

3017.56



Single use washing bowl in cellulos Packaging: 100 units

3017.57



ir wash basin in ABS for ampooing and hair care.



3017.58



Foot wash basin in ABS for foot care and hygiene of ankles and neels.